## HTM 01-05 Testing & **Validation Schedule Chart**

| The Dentisan Solution                           |               |                    |  |   |   |   |                 |  |  |  |  |
|---|---------------|--------------------|--|---|---|---|-----------------|--|--|--|--|
| HSD Codes                                       |               |                    |  |   |   |   |                 |  |  |  |  |
| 1129015   | 1129016       | 1129017            | 1129668  | 1129019   | 1129020   | 1129021   |                 |  |  |  |  |
| Wash-Checks® Red desappears when claim Location | V sh-cheeks"  |                    | CROSS-CHECKS Supergrammentalities, statistics county (2004) (2004 | STEAM ISO 11140-1 CLASS 5  STEAM CHART 121/134°C Sterilee, LOT 2:013-03 | EMU-GRAPH" 4 191 CIDTO 1.28 MARTER  CARGO CONTROL OF THE CONTROL OF T | EMU-GRAPH™ A (Adhesive Bucked) CLASS 6  STEAM  ©0700000 35 min-134 c dentisan |                 |  |  |  |  |
| Wash<br>Checks                                  | Wash<br>Check | Wash<br>Checks Pro | Class 4<br>Indicators  | Class 5<br>Indicators   | Class 6<br>Indicators   | Class 6<br>Indicators   | Heli Che<br>and |  |  |  |  |

|  |  | pack size: 50 | Holder pack size: 1 | pack size: 25 | pack size: 250 | pack size: 250                        | pack size: 250 | (ADHESIVE BACKED)<br>pack size: 250 | Challenge Device  1 Helix + 250 tests | 134°C<br>(for 3.5 mins)<br>Holder + 31 tests | 1 bag x 30 tests | 15 tests |
|--|--|---------------|---------------------|---------------|----------------|---------------------------------------|----------------|-------------------------------------|---------------------------------------|--|------------------|----------|
| Test   | Frequency  |               |                     |               |                |                                       |                |                                     |                                       |  |                  |          |
| Ultrasonic Cleaners                                      |  |               |                     |               |                |                                       |                |                                     |                                       |  |                  |          |
| Protein Residue Test                                     | Weekly   |               |                     | •             |                |                                       |                |                                     |                                       |  |                  |          |
| Cleaning Efficacy Test                                   | Quarterly / according to manufacturer instructions | •             | •                   |               |                |                                       |                |                                     |                                       |  |                  |          |
| Washer Disinfectors                                      |  |               |                     |               |                |                                       |                |                                     |                                       |  |                  |          |
| Protein Residue Test                                     | Weekly   |               |                     | •             |                |                                       |                |                                     |                                       |  |                  |          |
| Cleaning Efficacy Test                                   | Quarterly / according to manufacturer instructions | •             | •                   |               |                |                                       |                |                                     |                                       |  |                  |          |
| Autoclaves   |  |               |                     |               |                |                                       |                |                                     |                                       |  |                  |          |
| Automatic Control Test                                   | Daily  |               |                     |               |                |                                       | •              | •                                   |                                       |  |                  |          |
| Steam Penetration Test For vacuum autoclaves (N&S Class) | Daily  |               |                     |               |                |                                       |                |                                     | •                                     | •  | •                | •        |
| Cycle Verification Test                                  | Consult autoclave manufacturer                     |               |                     |               | •              | For handpiece<br>steriliser (eg. DAC) | •              | •                                   |                                       |  |                  |          |

1129018

Heli Check Indicators

and Process

Bowie Dick | Bowie Dick

**Starter Kit** 

## **Validating Performance**

Maintaining decontamination equipment at peak performance, not only enables you to meet regulatory requirements, but also means you can use the equipment with complete confidence. Testing and validation is a vital part of every practice's decontamination procedures and now, Dentisan can help you monitor performance by supplying a range of products to help carry out regular and reliable testing and validation of washer disinfectors, ultrasonic cleaners and autoclaves.

www.dentisan.co.uk

To place your order please call: 08700 10 20 43





1129667

**Bowie Dick** 

Green Card T









